

# Light Indicator Guide

- The reusable chip includes a light to indicate battery and pairing status

Color	Description	Next Steps
Green	<ul style="list-style-type: none"><li>▪ Successful insertion of chip into sensor</li></ul>	<ul style="list-style-type: none"><li>▪ Follow app instructions for pairing</li></ul>
Blue	<ul style="list-style-type: none"><li>▪ Successful pairing of chip to mobile device and app</li></ul>	<ul style="list-style-type: none"><li>▪ Verify sensor attachment so host device can receive data</li></ul>
Purple	<ul style="list-style-type: none"><li>▪ Battery seal tab has not been removed to activate battery</li><li>▪ Battery is obstructed</li></ul>	<ul style="list-style-type: none"><li>▪ Remove tab to activate battery</li><li>▪ Refer to Directions for Use for the Radius PPG Wireless Sensor</li><li>▪ Disconnect reusable chip from sensor, wait 30 seconds, insert chip into sensor</li></ul>
Orange	<ul style="list-style-type: none"><li>▪ Low sensor battery</li></ul>	<ul style="list-style-type: none"><li>▪ Consider replacing sensor, do not discard reusable chip</li></ul>
Red	<ul style="list-style-type: none"><li>▪ Depleted sensor battery</li><li>▪ Hardware or sensor failure, chip blinking board failure code</li></ul>	<ul style="list-style-type: none"><li>▪ Replace sensor, do not discard reusable chip. If issue persists, replace reusable chip.</li><li>▪ Contact Masimo Technical Support, or replace sensor and chip</li></ul>

